
Directions for Use (Single Patient Use Only) • Mode d'emploi (utilisation pour patient unique seulement) • Gebrauchsanweisung (nur bei einem einzigen Patienten anwenden) • Instrucciones de uso (únicamente para el uso de un solo paciente) • Istruzioni per l'uso (esclusivamente monopaziente) • Instruções de utilização (apenas para utilização por um único doente) • 使用説明 (僅供一位患者使用) • 使用説明書 (患者一人用)

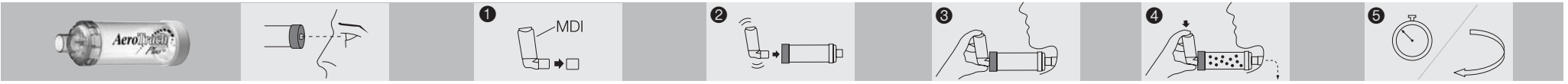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Manufactured by:
Trudell Medical International*
725 Third Street, London, Ontario,
Canada N5V 5G4
☎ +1 (519) 455-7060
☎ +1 (519) 455-7858
✉ customerservice@trudellmed.com
www.trudellmed.com

Authorised European Representative:
Trudell Medical International Europe Limited
Bicocity Nottingham, Pennyfoot Street,
Nottingham, United Kingdom NG1 1GF
REGISTERED
ISO 13485
QUALITY SYSTEM

AeroTrach Plus*
Anti-Static Valved Holding Chamber

Printed in Canada. 101061-001 Rev F. * trade marks and registered trade marks of Trudell Medical International. Vital-Oxide® is a trade mark of Vital Technologies. Copyright © 1999, 2000, 2007 – 2009 Trudell Medical International. All Rights Reserved. Patents pending.

WARNING: Remove device if patient shows signs of difficulty.
Intended Use: The **AeroTrach Plus*** VHC is intended to be used by patients who are under the care or treatment of a health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most MDIs prescribed by a health care professional or physician. The intended environments for use include the home, hospitals and clinics.

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Cautions ENG
AeroTrach Plus® VHC is designed for use with 15mm tracheostomy tubes with spontaneously breathing patients only.
 This device is designed as a slip fit onto the tracheostomy tube connection. It must be held in place during administration of medication.
 Do not spray more than ONE PUFF into the chamber at a time.

Notes
 VHC = Valved Holding Chamber
 Ensure directions for use have been read prior to use and are kept available at all times.
 If you have questions about the performance or usability of this product, please contact your healthcare provider.
 Do not leave the chamber unattended with children.
 Do not clean in dishwasher, boil or autoclave.
 This product contains no latex.
 Product should be replaced after 3 – 4 months of use. Environmental conditions, storage and frequency of use can affect product life span.
 When not in use, store in suitable container.
 Do not share.
 This device does not deliver supplemental oxygen.

Instructions for Disinfection
 After the clean water rinse, the following may be safely used: Enzyme Solvents, Vital-Oxide®, 1 part bleach/10 parts water solution. These should be soaked for 30 minutes followed by a clean water rinse.

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